


# Medicart - General specifications

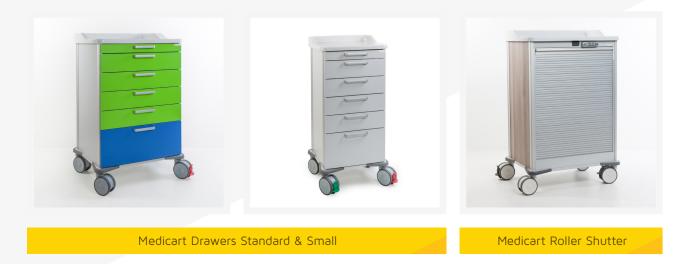

All three models of the Belintra Medicarts are **multifunctional medication carts**, assembled in a modular way, with numerous options in configuration and applications. The medication carts have a **very low rolling resistance** and today find their application in emergency, residential and care centers, anesthesia, surgery, ...

The Medicarts are built to the **highest safety standards** and also meet all ergonomic and hygienic requirements. They can be equipped with a wide range of accessories and **with or without IT and locking**.



### Synthetic worktop

The sleek synthetic worktop consists of **one continuous surface** with raised edges on 3 sides. There are **no sharp edges**, which makes the worktop easy to clean. The worktop has a high scratch resistance, suitable for all common cleaning products and disinfectants and is UV- and heat-resistant. Everything wipes right off, even disinfectants that stain like ISO Betadine<sup>®</sup>.

A stainless steel worktop is also possible upon request.



## Corner profiles

The extrusions are designed as such, that all Belintra accessories can be installed very easily, even by your own technical staff.






#### Castors

The castors meet the highest quality standards, and the design is the result of years of expertise. Studies have shown that the dual castors result in **50% less rolling resistance and improved maneuverability of the cart**.

Images for illustrative purposes only, not binding. Specifications are subject to change without notice.



The Medicart has dual Tente® swivel castors, 1 with a brake (antistatic).

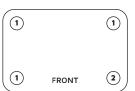

Swivel castor Swivel castor with brake - antistatic 1. 2.

Option: swivel castor with brake and/or swivel castor with directional lock

#### Double castors study: low rolling resistance and easy to manoeuvre

Type of medical cart		Competitor cart 1	Competitor cart 2	Belintra Test cart
Start forward thrust Castors straight	Measurement 1	5,38	7,90	2,79
	Measurement 2	6,40	6,30	3,98
	Measurement 3	7,40	6,90	3,92
	Average	6,39	7,03	3,56
Start forward thrust Castors transverse	Measurement 1	8,30	10,20	7,70
	Measurement 2	8,10	9,50	7,40
	Measurement 3	9,80	12,40	6,10
	Average	8,73	10,70	7,07

Images for illustrative purposes only, not binding. Specifications are subject to change without notice.