



Drying Instructions

Sterisystem™ Closed Stainless Steel Case Carts



Important – Please pass these instructions on to the people responsible for the cleaning of the Sterisystem™ Closed Stainless Steel Case Carts

For the optimum performance of your case cart make sure to follow the drying instructions below:

- Drying time: 8 minutes at 85°C in dry air.
- Dry air = air with a relative humidity of maximum 60% at an ambient temperature of 21°C. Converted this is a relative humidity of maximum 1,1% at 85°C.

For additional information or questions about your Sterisystem™ Closed Stainless Steel Case Cart please contact your local Account Manager.