




Before any use, connect the Roll-IT using the coiled lead included with it to the mains and leave the battery to charge for at least 8 hours without interruption before first use.

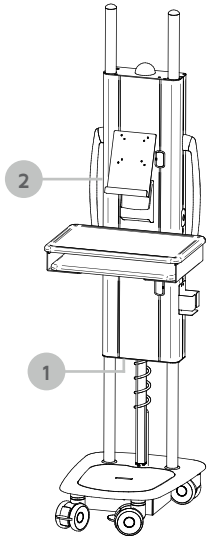

1. The built-in PC can be started up with the power switch located on top of the Roll-IT. After the operating system has been turned off, the PC can be restarted using the on-off switch.

NOTE: The LED indicator remains active even when the switch is turned off. The LED indicator will only go out if the Roll-IT is in sleep mode due to low battery.

2. The height of the Roll-IT can be adjusted electrically. The high-low switch is on the left side of the device.



Take care to ensure that the trolley does not crush itself, other persons or objects when you adjust the height.

» Battery status indicator

The low voltage system in the Roll-IT indicates the battery status using an LED indicator on the top of the device, and the Battery Monitor application on the desktop of the PC (only if the application has been installed, not applicable to a thin client, among others).



To open the Battery Monitor application:

1. Navigate to the system bar at the bottom right and right-click the icon
2. "Show" opens the application
3. Select "Always on top" in order to keep the application in the foreground
4. "Exit" closes the application



The remaining charge in the battery is between 100% and 70%.

The LED flashes when the Roll-IT is not plugged in to the mains supply. **Leave the battery to charge as often as possible until the LED indicator stops flashing.** The battery is then 100% charged.



The remaining charge in the battery is between 69% and 40%.



The remaining charge in the battery is between 39% and 10%.

Repeated use of the Roll-IT with a battery charge of 30% or less reduces the life of the battery considerably.



The remaining charge in the battery is less than 10% when the LED is flashing red.

Plug the Roll-IT immediately into the power outlet and leave the battery to charge to 100%!



When the battery charge has fallen to 5%, this pop-up will appear. A few minutes later, the PC will shut down and the LED indicator will go out. The Roll-IT has gone into sleep mode.

Plug the Roll-IT immediately into the power socket and leave the battery to charge to 100%!

» Important guidelines for the daily use of Roll-IT

- Adapt your speed to the immediate circumstances when moving the trolley. To retain control over the trolley in unexpected situations, it should never be moved at more than normal walking pace (4 to 5 km/h).
- Use the plastic handles to move the trolley. Never move the trolley by means of the aluminium guides.
- Ensure that there are no stray objects in the way while moving the trolley.
- The maximum load on the worktop is 25 kg.
- The maximum load on the pull-out sliding keyboard tray is 10 kg.
- Ensure that the cable connector has been unplugged before you move the Roll-IT. Never pull on the lead. Hold the plug itself rather than the lead when disconnecting the mains lead.
- **Plug the Roll-IT into the mains supply as often as possible. Charge the battery up to 100% each day.**
- Before moving the trolley, check that the brakes on its wheels have been released.

For complete instructions, consult the Instructions for Use supplied with the trolley or go to www.belintra.com/en/downloads.